

Bi o ɛ le lo sùwàbù olomi

1

Sùwàbù alawọ búlù ninu túbùù òniké funfun meji

Apo ọlọra funfun meji, ti akọlẹ “VRD” ati “PHL” wa lara wọn

Apoti onipaali ti akọlẹ UKHSA URGENT MEASLES TESTING wa lara rẹ

Àwọn Àkóónú

UK Health Security Agency
Dear (Name of case)
Your doctor has had suspected diagnosis control of infectious mumps and submit to confirm the diag to prevent spread
We can test for my sample is taken in young children. T
As the UKHSA to collect the samp these immediat
The kit you have local hee department (V completed test
1. For one to use: remove the clear tube (A) back inside the clear tube (B), and replace the white cap. Please print name, date of birth, today's date on the label on the clear tube.
2. Place the labelled tube containing the swab inside the self-adhesive clear plastic bag. Push the air out of the bag. Remove the rest of the kit.
3. Repeat steps 1-2 for the second swab.

For submitting the only: UKHSA name

MR: Yes No Date:
MM: Yes No Date:
MR: Yes No Date:

Fọ́ọmu ibeere

Apo ọlọra funfun

2

2 mins

X2

Yọ sùwàbù alawọ búlù kuro ninu paali. Ra sùwàbù alawọ búlù lori ilà erigi fun iseju meji.

3

X2

4

Tun Igbesẹ 2 ati 3 ɛ

Fi sùwàbù alawọ búlù tutu pada sinu túbùù òniké funfun ki o si fi ideri miran de. Kọ orukọ alaisan, oḣo ibi rẹ, oḣo oni sori túbùù kọḣkan.

5

VRD

Fi túbùù akọkọ sinu apo ọlọra funfun ti VRD wa lara rẹ. Fú-n atẹgun jade, yọ edidi pupa kuro lati oke ki o si lẹ ẹ pa.

6

PHL

Fi túbùù keji sinu apo ọlọra funfun ti PHL wa lara rẹ. Fún atẹgun jade, yọ edidi pupa kuro lati oke ki o si lẹ ẹ pa.

7

UKHSA URGENT MEASLES TESTING

Yẹ ẹ wò ki o si pari fọ́ọmu ibeere ki o si fi sinu apoti onipaali pelu apo ọlọra funfun meji toni awọn sùwàbù ninu ki o si páá dé.

8

UKHSA URGENT MEASLES TESTING

Fi apoti onipaali sinu apo ọra funfun, lẹ ẹ pa ki o si da pada fun aferujisẹ tabi akọsọmọḣe itoju ilera.